

How to use evercare® Foam Dressing

REF: EFD050050, EFD050050U, EFD065065UX, EFD100090UX, EFD100100, EFD100100U, EFD100100B, EFD150150, EFD150150U, EFD150150B, EFD200100, EFD200100U, EFD200200, EFD200200U, EFD210170UH.

evercare® Foam Dressing Border / Non Border may be used by lay person under supervision of health care professionals. Secondary dressing is necessary when using evercare® Foam Dressing Non Border as it has no attaching border.


1

Cleanse the wound in accordance with clinical procedure.

2 Pouch opening

Select an appropriate dressing size/shape so that the foam pad overlaps the wound margins by approximately 2 cm.

Cut the dressing to proper size, if necessary.

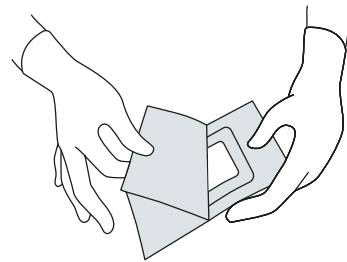


3

A Foam Dressing non border / with border, self-adhesive

Remove one piece of the release liner.

Place the foam side directly to the wound surface and remove the other release liner.

Gently secure the dressing to the skin.

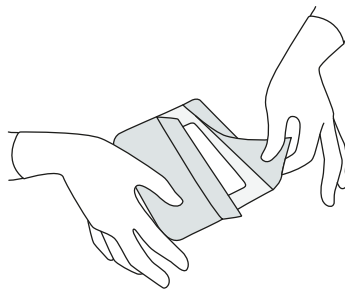

B Foam Dressing non border for elbow / heel

Place the white foam side directly to the cleaned wound.

Secure the foam dressing with a bandage or other fixation.

