

# How to use evercare® Medical Hydrogel Dressing

REF: REFEGH050050N, EHG075045N, EHG100100B, EHG100100N, EHG150150B, EHG150150N, EHG200100N.

evercare® Medical Hydrogel Dressing may be used by lay person under supervision of health care professionals.

1

After the wound is cleaned and disinfected, product of certain specification is chosen to the wound area, the hydrogel dressing should be 5–10 mm larger than the wound in margins.

2

## Pouch opening

Take out the sterile dressing from the package.



3

**Hydrogel Adhesive Dressing:**  
Peel off one half of the white plastic liner, position it on the wound or skin, smooth onto the treated place and peel off the remaining half of plastic liner.

**Hydrogel Non-Adhesive Dressing:**  
For treating larger wound areas, several dressings may be used side by side. For smaller areas/wounds, dressing may be cut to proper size.



4

**Hydrogel Adhesive Dressing:**  
When the dressing is attached to the place, the backing films must be removed.

**Hydrogel Non-Adhesive Dressing:**  
Secure the applied Hydrogel Non-Adhesive Dressing with a suitable secondary dressing, bandage, tape, or compression system where applicable.

