

How to use evercare® Silicone Foam Dressing

REF: ESF075075F, ESF100100F, ESF125125F, ESF150150F, ESF200100F, ESF200150F, ESF175170FS, ESF230230FS, ESF050050L3, ESF125050L3, ESF120060L3, ESF075075L3, ESF100100L3, ESF150150L3, ESF050050N, ESF100100N, ESF125125N, ESF150150N, ESF200100N, ESF200130NH, ESF200200N.

evercare® Silicone Foam Dressing **Border** / **Border Lite** / **Non Border** may be used by lay person under supervision of health care professionals.

1

Cleanse the wound according to clinical practice.

Dry the surrounding skin thoroughly.

2 Pouch opening

Select an appropriate dressing size.


Border: The non-bordered dressing part should cover the wound with at least 1–2 cm wider margin.

Border Lite / Non Border: The wound pad should cover the surrounding skin by at least 1–2 cm wider margin.



3

Remove first the bottom release film and apply the adherent (white) side to the wound.

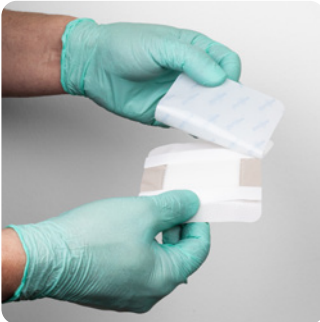

4

Remove the remaining release film(s) (Border Lite: liners) and smooth down the border on the skin. Do not stretch the dressing.

The dressing can be covered by other compression bandages.

