

# How to use facemask-with-tie-bands

REF: 2884, 2885

1

Ensure your hands are thoroughly cleaned.



2

Place the mask on your face and make sure to cover your nose and mouth.



3

Tie the upper tie-bands at the crown of your head. Tie the top tie-bands of the mask before tying the bottom tie-bands.



4

Press the nose wire and unfold the mask carefully to ensure that it fits your nose and cheeks.



5

Change the facemask when contaminated and as often as your local recommendations.



6

Remove the mask carefully. Don't touch the front of the mask which may be contaminated. Untie the bottom tie-bands first and then the upper tie-bands carefully. Remove it without the mask landing on your clothing or skin, hence less likelihood of further contamination.



7

Dispose the mask by holding the tie-bands.

