



Priming evercare® inLine, vented


Priming recommendation to prime evercare® inLine infusion set when using semirigid container or infusion bag.

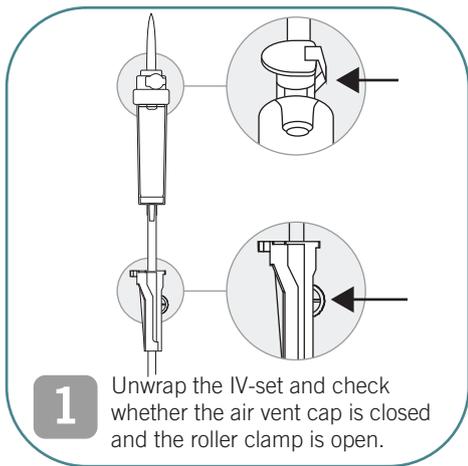


1 Unwrap the IV-set and check whether the air vent cap is closed and the roller clamp is open.

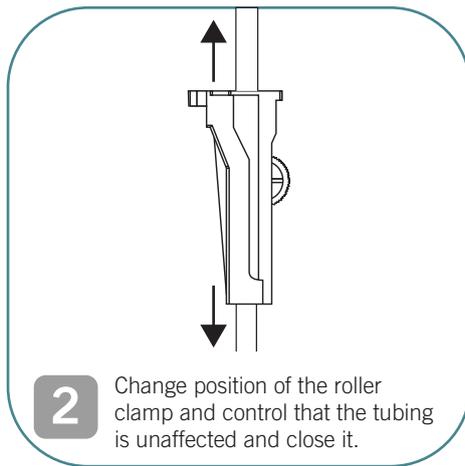


2 Change position of the roller clamp and control that the tubing is unaffected and close it.

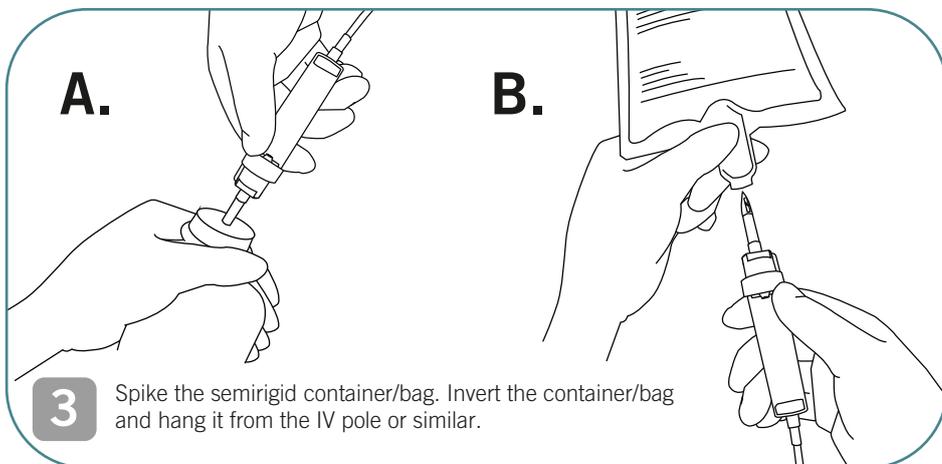


3 Spike the semirigid container/bag. Invert the container/bag and hang it from the IV pole or similar.

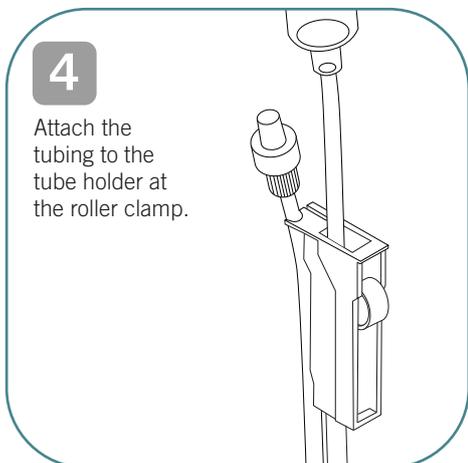


4 Attach the tubing to the tube holder at the roller clamp.

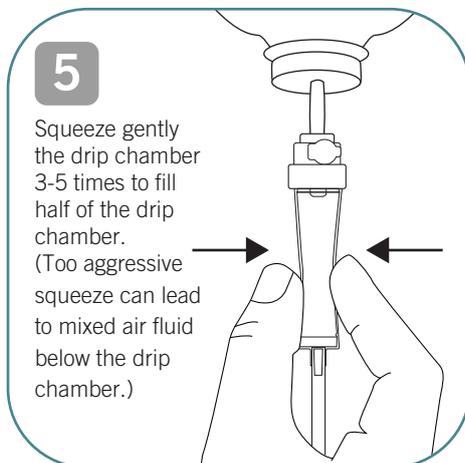

5 Squeeze gently the drip chamber 3-5 times to fill half of the drip chamber. (Too aggressive squeeze can lead to mixed air fluid below the drip chamber.)

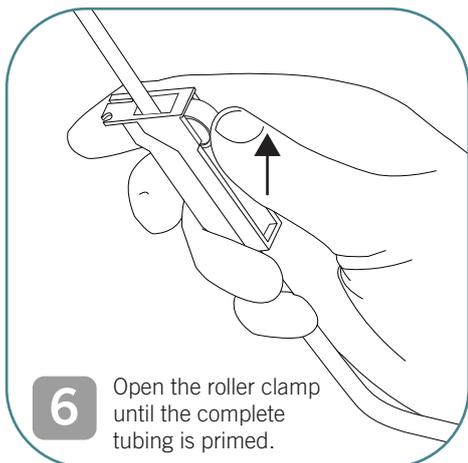

6 Open the roller clamp until the complete tubing is primed.

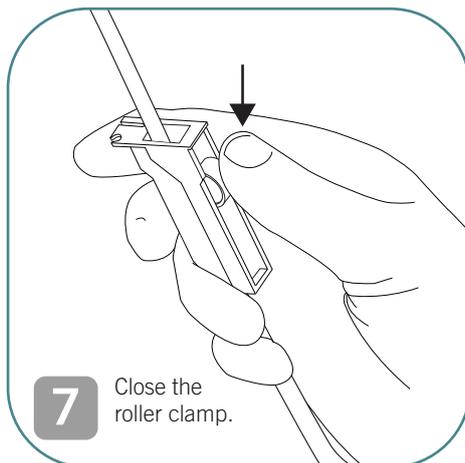

7 Close the roller clamp.

REF

10499	10590
10500-01	10591
10501-01	10700
10502-01	10701
10503-01	10702
10570-01	10703
10504-01	10704
10505-01	10705
10506-01	10706
10577-01	10707
10578-01	10708
10579-01	10709
10580-01	10710
10581-01	10711
10582-02	
10585-02	

Contact person:

! See the package for complete IFU instructions for use, this is only a guide for users !

DO NOT OVER-TIGHTEN connections. **DO NOT USE** any instruments to tighten connections

Tip:
Glass and non fully collapsible containers: Keep the vent closed during spiking and priming. Only open the vent flap when venting is required.

evercare®
medical

We care more

www.evercare-medical.com