



evercare® inLine monoset


With Needle Free Connector

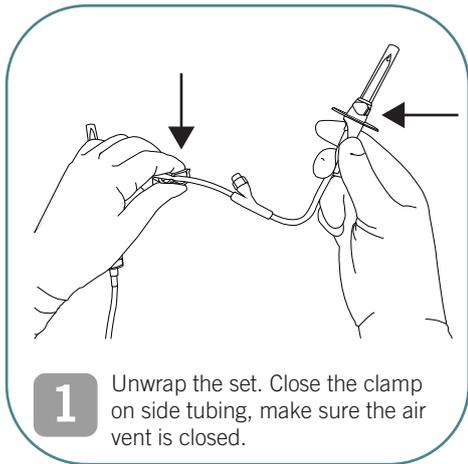


1 Unwrap the set. Close the clamp on side tubing, make sure the air vent is closed.

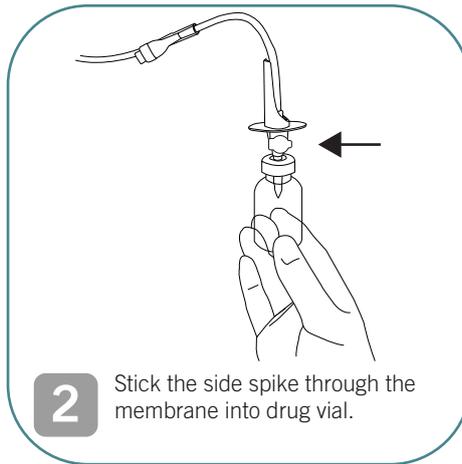

2 Stick the side spike through the membrane into drug vial.

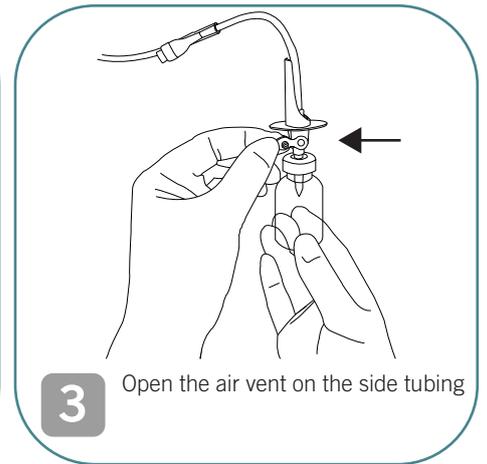

3 Open the air vent on the side tubing

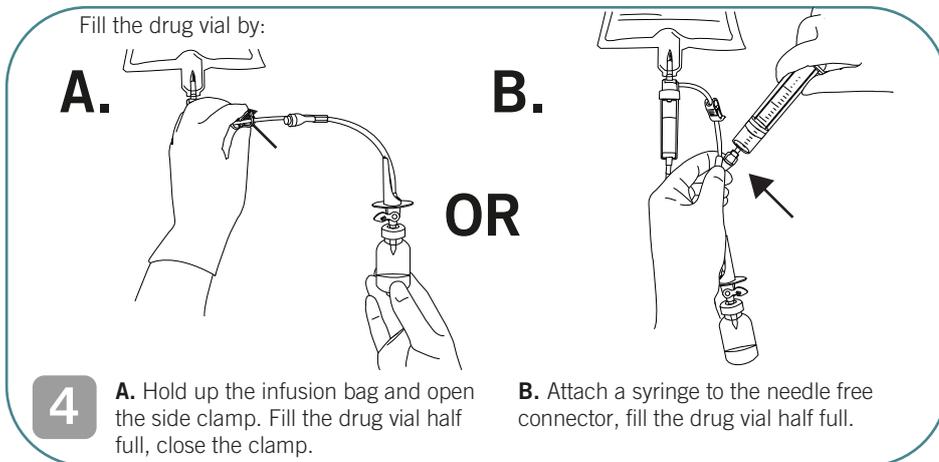

Fill the drug vial by:


A.

B.


OR

4 **A.** Hold up the infusion bag and open the side clamp. Fill the drug vial half full, close the clamp.


B. Attach a syringe to the needle free connector, fill the drug vial half full.

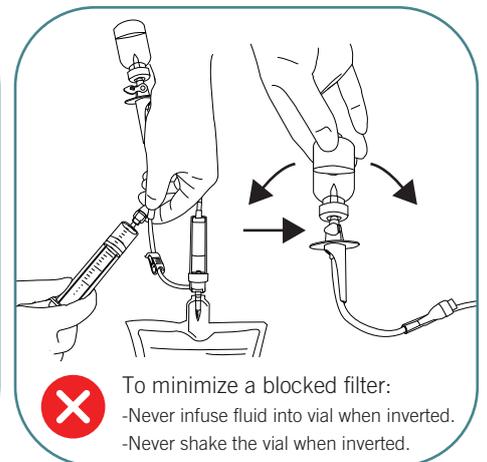


X To minimize a blocked filter:
-Never infuse fluid into vial when inverted.
-Never shake the vial when inverted.

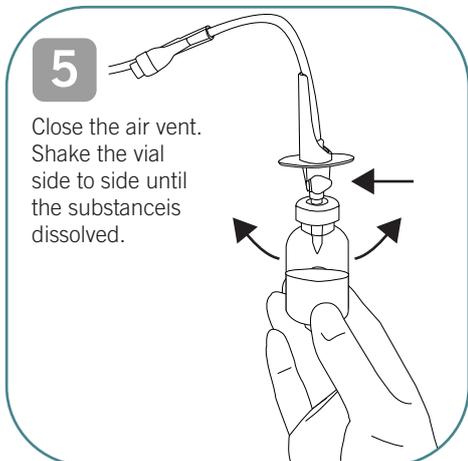

5 Close the air vent. Shake the vial side to side until the substance is dissolved.

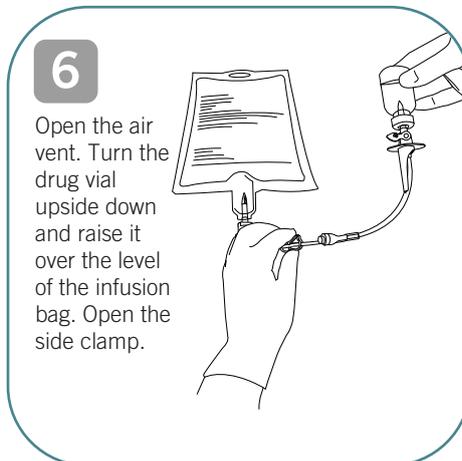

6 Open the air vent. Turn the drug vial upside down and raise it over the level of the infusion bag. Open the side clamp.

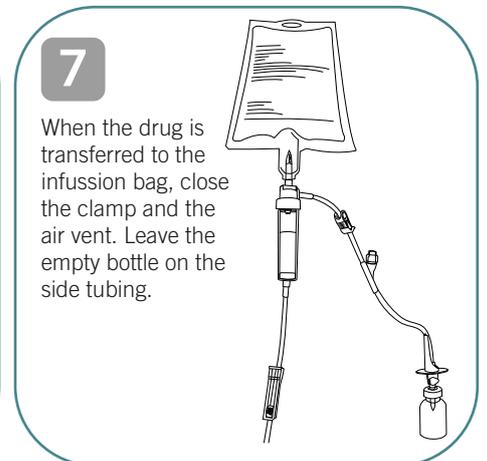

7 When the drug is transferred to the infusion bag, close the clamp and the air vent. Leave the empty bottle on the side tubing.

! See the package for complete IFU instructions for use, this is only a guide for users !

DO NOT OVER-TIGHTEN connections. **DO NOT USE** any instruments to tighten connections

Contact person:

REF

10515-01
10516-02
10517-02
10518-02
10576-01
10583-02

evercare®
medical
We care more
www.evercare-medical.com