

How to use SELEFA® SENSE LIGHT medical examination gloves


REF: 100018, 100019, 100020, 100021, 100022

Please note! Wash your hands before donning the gloves. Don't use the gloves with long sharp fingernails or rings. If your hands get contaminated during doffing, wash your hands, or use an alcohol-based hand sanitizer.

Opening the dispenser box


1

Turn the dispenser box upside down and press down the opening at the perforated cut.

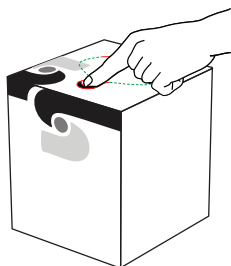


2

Grab the loose opening part with your thumb and index finger and remove the loose part.

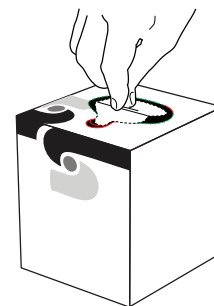


3

In the opening, make sure the cuff comes out first.

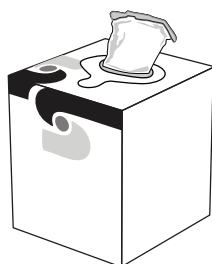

4

Place the dispenser box in holder with opening facing down. Grab the gloves only by the cuff.

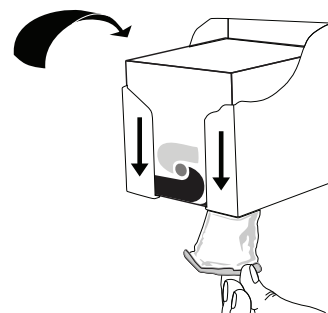

Donning


1

Before donning, ensure the gloves are free from pinholes or tears. Insert your hand in the glove ...




2

... and pull the glove firmly so that it fits comfortably.

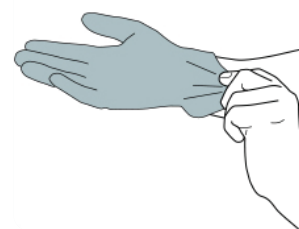

3

Check that the gloves fit well.

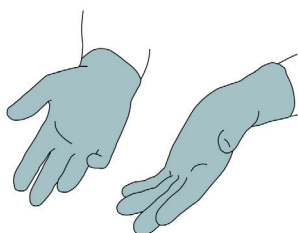

How to use SELEFA® SENSE LIGHT medical examination gloves

REF: 100018, 100019, 100020, 100021, 100022

Please note! Wash your hands before donning the gloves. Don't use the gloves with long sharp fingernails or rings. If your hands get contaminated during doffing, wash your hands, or use an alcohol-based hand sanitizer.

Doffing

1

Grasp the outside of the cuff of the glove without touching the skin to avoid contamination.




2

Peel off, holding the glove in the gloved hand. Allow the glove to turn inside out.




3

Place the removed glove in your gloved hand. Slide your fingers of the ungloved hand under the remaining glove at the cuff ...

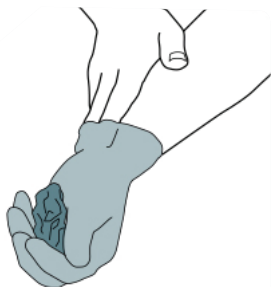

4

... and peel off the second glove over the first glove.

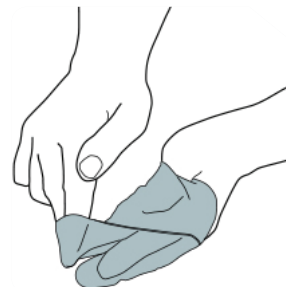

5

Discard the gloves.

