

Priming evercare® inLine, unvented

REF: 10507-02, 10508-01, 10509-01, 10510-01, 10511-01, 10512-01, 10513-02, 10514-01, 10572-01, 10597, 10598, 10599.


Priming recommendation to prime evercare® inLine infusion set when using semirigid container or infusion bag.

Never use a glass bottle with unvented infusion set.

Please note! See the package for complete IFU instructions for use, this is only a guide for users. **DO NOT OVER-TIGHTEN** connections. **DO NOT USE** any instruments to tighten connections. Tip: Glass and non fully collapsible containers: Keep the vent closed during spiking and priming. Only open the vent flap when venting is required.

1

Unwrap the IV-set and check that the roller clamp is open.

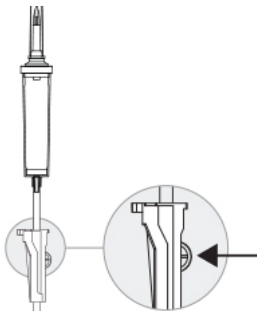


2

Change position of the roller clamp and control that the tubing is unaffected and close it.




3

Spike the semirigid container/bag. Invert the container/bag and hang it from the IV pole or similar.

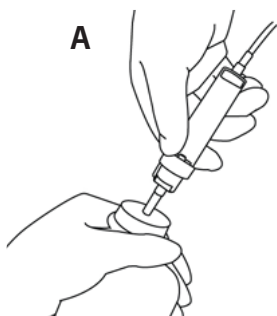


B

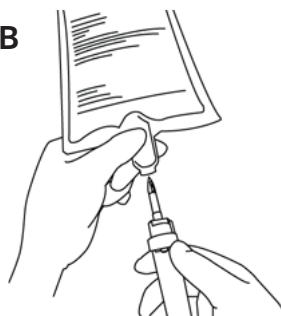


4

Attach the tubing to the tube holder at the roller clamp.

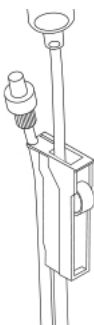

5

Squeeze gently the drip chamber 3–5 times to fill half of the drip chamber. (Too aggressive squeeze can lead to mixed air fluid below the drip chamber.)

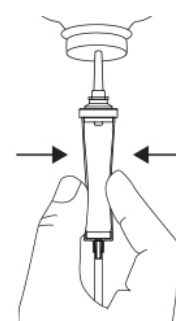

Priming evercare® inLine, unvented


REF: 10507-02, 10508-01, 10509-01, 10510-01, 10511-01, 10512-01, 10513-02, 10514-01, 10572-01, 10597, 10598, 10599.

Priming recommendation to prime evercare® inLine infusion set when using semirigid container or infusion bag.
Never use a glass bottle with unvented infusion set.

Please note! See the package for complete IFU instructions for use, this is only a guide for users. **DO NOT OVER-TIGHTEN** connections. **DO NOT USE** any instruments to tighten connections. **Tip:** Glass and non fully collapsible containers: Keep the vent closed during spiking and priming. Only open the vent flap when venting is required.

6

Open the roller clamp until the complete tubing is primed.

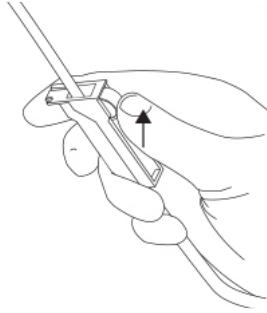

7

Close the roller clamp.

