
# Face Shield

**REF** 270100

**SIZE** 33 x 22 cm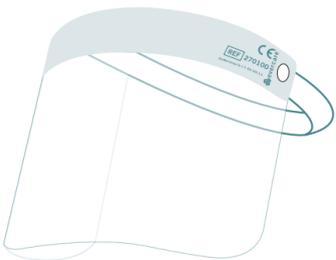

**de** evercare® Gesichtsschild

**fr** Visière faciale evercare®


**sv** evercare® skyddsvisir

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**da** evercare® ansigtvisir

**no** evercare® ansiktsskjerm

**nl** evercare® gelaatsscherm

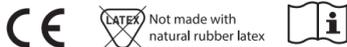Not made with natural rubber latex

**OneMed Group Oy 1 S -EN 166 3 S**

**OneMed Group Oy**  
Metsäläntie 20 FI-00320 Helsinki, Finland

FRONT REV 2 / 2020–12

**en**

## Description and Intended Use:

The evercare® face shield, REF 270100, is a clear, transparent disposable splash shield, made of an optical grade PET film with a contoured polyurethane foam headband for comfort and elastic strap (PET) for fixation. This product is intended to provide full face cover, in order to protect healthcare professionals from body fluids and other liquid splashes while performing a clinical, cleaning or laboratory procedure. The product is reusable (if disinfected), but the employer guideline of re-use shall prevail. It is recommended the product shall only be used by a single wearer, to reduce risk of cross-contamination.

Re-use of the face shield for multiple procedures is associated with a risk of contamination, therefore precaution measures must be taken according to employer guidelines and the recommended cleaning instruction enclosed within this IFU.

## User instructions:

1. Visually inspect the face shield to ensure that it is in good condition.
2. Use a marker pen to write the wearer's name on the head band, if desired.
3. Remove the protective film on the inner side of the shield.
4. Holding the face shield with both hands, place the elastic strap behind your head, making sure that the foam rests on your forehead. Try to avoid touching the PET-film with your hands.
5. Once the shield is situated, make sure that it covers the front and sides of the face and that no areas are left uncovered. Do not remove face shield until risk of exposure to liquid splashes is over.
6. Remove the face shield by lifting the elastic strap from the back of your head. Avoid touching contaminated areas. If using gloves, take them off before removing the face shield.
7. Disinfect the product according to cleaning instructions and per employer guidelines. If the face shield is exposed to possibly infectious materials during use (e.g. body fluids from an infected person) it is recommended to dispose the product immediately after use.
8. Dispose after use as per applicable waste handling guidelines.

## Warning:

- This Instruction for Use (IFU) shall be made available for all users of the product. The IFU is provided in the transport box, digital format available at [www.onemed.com/what-we-do](http://www.onemed.com/what-we-do).
- Appropriate disinfection method shall be applied between procedures to reduce risk of contamination, and procedure must comply with local employer guidelines.
- Do not share the same item between different wearers, to reduce risk of cross-contamination.
- Check product before use. Do not use if the product is damaged or does not fit, e.g. product is scratched, the elastic band is too slack to keep the shield situated, etc.
- The anti-fog treatment may lose its efficiency and fogging may occur under certain environmental conditions.
- The foam (polyurethane), when in contact with skin, may cause skin irritation to susceptible individuals. The materials are latex free.

## Cleaning:

The evercare® face shield is intended to be used by one wearer. If the employer authorizes re-use of face shields, the product shall be cleaned between procedures as per the employer cleaning guidelines. Recommended cleaning method:

1. Disinfect hands with sanitizer.
2. Remove the face shield by lifting the elastic strap from the back of your head.

3. Wash with detergents. Detergents with or without surfactants are allowed.
4. Spray the plastic parts of the face shield with appropriate disinfectant. Ethanol or isopropanol is recommended.
5. Wipe off with a clean cloth.
6. Disinfect hands with sanitizer after cleaning.

**Note:** Do not dip or soak the foam with alcohol or disinfectants as it will damage the foam. The product has been tested to withstand cleaning at least 20 times, with no indication of product deterioration.

## Storage:

Store the product in a dry place at 5– 25°C.

Keep dry and away from sunlight.

Shelf-life: 3 years (stored in proper conditions in its original packaging).

## Compliance:

The product is in compliance with Personal Protective Equipment Regulation (EU) 2016/425. The PPE is categorized as a category II, and conforms to harmonized standard EN 166:2001 Personal eye-protection specification.

EU type conformity assessment (Module B) performed by RISE Research Institutes Sweden AB, Box 857, SE-50115 BORÅS, Notified Body nr. 0402. Testing has been performed according to EN 166:2001.

Product marking "OneMed Group Oy 1 S -EN 166 3 S" declares the face shield is assessed according to EN 166. The face shield is of optical class 1 (class 1 out of 3, where 1 represent the highest class) with increased robustness (S) and its area of use is (3) - protection against liquid splashes.

## Declaration of Conformity:

The Declaration of conformity and Instruction for Use is available at [www.onemed.com/what-we-do](http://www.onemed.com/what-we-do).

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